




Sampling Kit Instructions

The following items are included in your allergy test sampling kit:

- 2 lancets to prick finger (single use only, one spare included)
- 1 blood sample collection tube
- 1 safety transport tube
- 1 antiseptic wipe
- 1 plaster
- 1 waste bag
- 1 return box container
- 1 pre-addressed (freepost) envelope

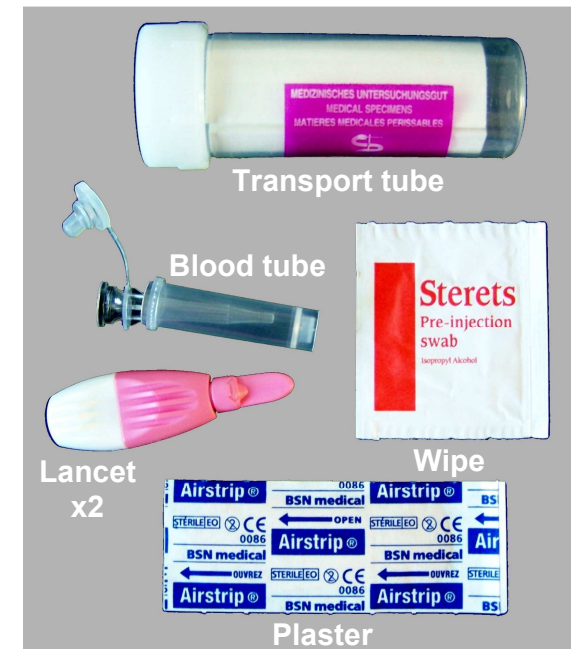

- The purpose of this kit is to self-administer the collection of a blood sample for your allergy test. Please read through the instructions before attempting to use the test. There are 14 instruction points in total.
- This kit contains a sharp implement (Lancet) which may cause injury if mishandled or used inappropriately.
- Store kit at room temperature before and after usage.

Safety Advice

- **Keep kit contents out of reach of children.**
- Kits should be used by adults or by children under **adult supervision only**.
- Lancets are for **single use only** and should not be shared with others.
- Handle blood with care.
- Once used the lancets and other materials in contact with blood could be infectious, dispose of safely as directed in the instructions.

